

**MONITEX**

# Algimax-II


GX300

## Alginate Mixer

Smooth mixture in seconds for a quick and accurate impression

Patented anti-vibration mechanism

Maximize workflow and hygiene

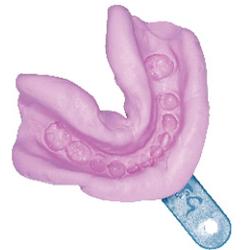

**MONITEX**

# Algimax-II GX300 Alginate Mixer

The patented ALGIMAX-II is a fully automated alginate mixer that standardizes the alginate mixing procedure. Enjoy smooth mixtures, bubble and lump free, to capture the most accurate impression. This high quality performance is made possible by the motors in the mixer to optimize the alginate setting reaction to just seconds, saving dentists and their assistants valuable time.



Simple steps to creating a perfect impression:



1. Add alginate powder and water into the mixing cup.

2. Moisten with the stirrer for a few seconds and close the mixing cup.






3. Put the mixing cup into the mixing well.

4. Cover the lid and press "START".



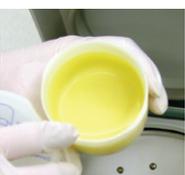

5. After mixing, use the spatula and remove the mixed impression.

### 3 Default Mixing Modes

Scoops	Mixing Time
☪	8 secs
☪ ☪	10 secs
☪ ☪ ☪	12 secs

### Specifications:

- Timer Range 1 ~ 16 seconds
- Mixing Modes 3 (Default: 8, 10, 12 secs)
- Speed 3,400 rpm
- Power Input AC 110 or 220V, 50/60 Hz
- Dimensions 250(L) x 205(W) x 285(H) mm
- Net Weight 17.5 kg
- Accessories Spatula/Stirrer/Storage Case/  
Water Dosing Bottle/Mixing Cups(x4)



**MONITEX** INDUSTRIAL CO., LTD

5F, 70, Sec. 1, Guangfu Rd., Sanchong Dist., New Taipei City 24158, TAIWAN  
 TEL: + 886 2 29993869 FAX: + 886 2 29993647  
 Email: monitex@monitex.com.tw  
[www.monitex.com.tw](http://www.monitex.com.tw)

