

BluTab® Waterline Maintenance Tablets Instructions

BluTab®
Because it works.

BluTab® Waterline Maintenance Tablets are formulated specifically for dental unit waterlines. BluTab is designed to be continuously present in your waterlines and helps keep lines clean.

Initial Treatment:

Before starting a treatment program with BluTab®, shock lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When shocking dental unit waterlines, always follow a recognized line shocking/cleaning protocol. Repeat shocking to clean lines every 2 months or more frequently when water quality results indicate. For a list of recommended shock products, call 888.843.3343.

Directions for Use:

1. Place tablet or tablets (see table below) in empty dental unit water bottle every time you refill the water. BluTab is intended for use with potable water (including tap or distilled).
2. Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.

Water Volume	750mL Tablets
700 - 750 mLs	1
1500 mLs	2
2L or 2000 mLs	3

Water Volume	2L Tablets
2L or 2000 mLs	1
4L or 4000 mLs	2

3. BluTab may be left in the waterlines overnight and for extended periods of non-use. BluTab has been shown to be effective for up to 28 days. Use water solution until it is expended; or up to 28 days. It is not necessary to purge lines at the end of each day.

Clinical monitoring of water quality can ensure that procedures are performed correctly. Test water quality quarterly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat *Initial Treatment*.

Precautionary Statement:

Caution: Eye: Tablet may cause irritation to the eyes (diluted product is non-irritating). Avoid contact with skin, eyes or clothing. Skin: Prolonged contact may cause irritation and redness. Wash hands with soap and water after handling.

First Aid - Eyes: Immediately flush eyes with water for at least 10 minutes. If irritation persists, get medical attention.

Skin: For skin contact flush with plenty of water. If irritation persists, get medical attention. Take off any contaminated clothing; wash before reuse.



THEIR PROTECTION
YOUR ASSURANCE

Questions?

888.843.3343 | support@proledgedental.com
7042 S Revere Parkway, Suite 400, Centennial, CO 80112

©ProEdge Dental v5