

INSTRUMAX[®]

PINK

Instrument Grade Disinfectant – LOW LEVEL



Reduce the risk of bacterial
cross infection by using
Instrumax[®] Pink

INSTRUMAX[®]

PINK

Instrument Grade Disinfectant – LOW LEVEL

- **Specially formulated instrument grade disinfectant – low level**
- **Suitable for numerous applications**
- **Kills a broad range of pathogenic bacteria**

INSTRUMAX[®] PINK is approved for use on Non Critical Medical Devices including stethoscopes, thermometers, blood pressure cuffs, tourniquets, humidicribs, MRI machines and nebuliser face masks commonly used in the treatment of asthma.

INSTRUMAX[®] PINK is also suitable for a wide range of non medical applications including training masks and mannequins used for CPR training and other related applications. Any application where face masks are subjected to re-use and require on site disinfection may be suitable for **INSTRUMAX[®] PINK**.



Instrumax® Pink is proven to kill:

BACTERIA


- Acinetobacter;
- Escherichia coli;
- Enterococcus faecalis (VRE);
- Proteus vulgaris;
- Pseudomonas aeruginosa;
- Salmonella choleraesuis;
- Staphylococcus aureus (MRSA or Golden Staph)

VIRUSES

- Hepatitis B Group virus;
- Herpes Simplex virus;
- Influenza virus

FUNGI

- Candida albicans

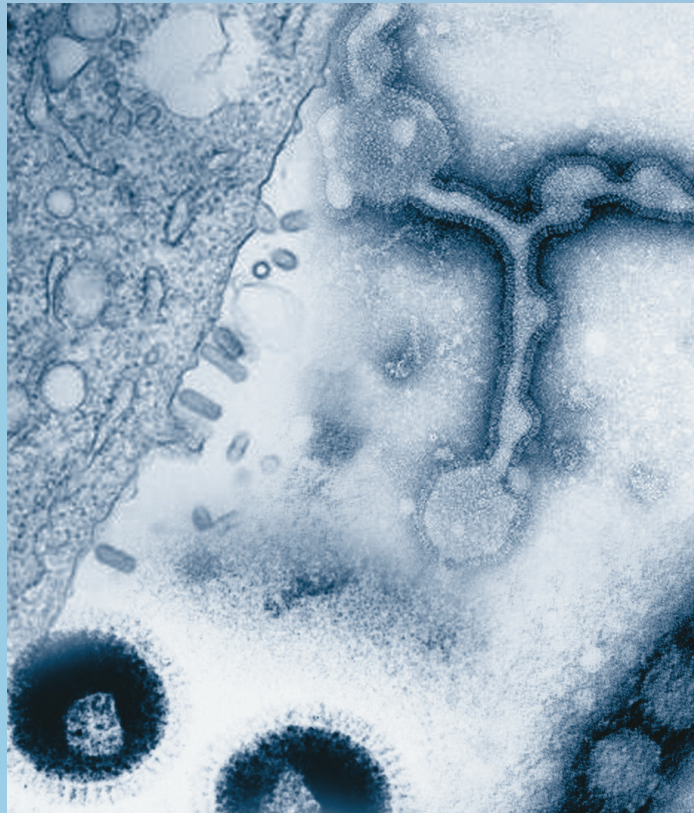

Directions For Use:

1. Thoroughly pre-clean the device or non-medical instrument in warm water with Neutral Detergent (Sonidet® or Matrix®) according to manufacturer instructions.
2. Apply **INSTRUMAX® PINK** liberally on to all parts to be disinfected and leave moist for 10 minutes OR soak the parts directly in the solution for not less than 10 minutes.
3. Rinse off thoroughly with clean running water and discard the used solution to the sink/sewer.
4. Do not re-use the solution. The use of gloves is recommended whenever using disinfectants.

INSTRUMAX® PINK is not intended to be used on Semi-Critical or Critical Medical Devices.

Please consult Technical Bulletin prior to first use. For information on Technical Bulletin and Safety Data Sheet please go to www.whiteley.com.au or Phone 1800 833 566.

HEALTH STATEMENT

Whiteley Medical makes every effort to ensure that its products are always safe to use, however products can affect individuals differently and the company cannot predict individual responses. Therefore careful reading of the Instrumax Pink® Safety Data Sheet is essential prior to first use.

ENVIRONMENTAL STATEMENT

Whiteley Medical manufactures products that comply with Australian and International environmental policy. Whiteley Medical aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers.

INSTRUMAX® PINK is fully biodegradable and will not adversely impact aquatic or marine environments.

Cleaning and Disinfection of Instruments and Devices should be repeated after each use. **INSTRUMAX® PINK** is suitable for use with MedelNeb, Airssential or Rapidflo Nebulisers, and VapoTherm units.

Whilst we have had the product tested against a range of hard plastics, we cannot be certain that the disinfectant action will not affect other plastics or natural rubber. Overuse and prolonged soaking may lead to early deterioration of some plastic or rubber surfaces.



DISTRIBUTED BY:



ABN No. 24 000 906 678
PO Box 1076, North Sydney NSW 2059
Product Support Hotline: 1800 833 566
Email: whiteley@whiteley.com.au
www.whiteley.com.au

NZBN: 9429041224243
PO Box 22-519, Otahuhu, AUCKLAND 1640
Product Support Hotline: 0800 257 352
Email: whiteley@whiteley.co.nz
www.whiteley.co.nz